LadyBalance

CE  
0543

## Please read this leaflet carefully before use.

LadyBalance® is a prebiotic (supports own micro flora) which restores natural pH and maintains vaginal health.

**Intended use:** Reduces unpleasant vaginal odour and discharge associated with bacterial vaginosis (BV). Relieves vaginal dryness, itching, irritation and soreness.

**Instructions for use: *Treatment:*** Use one vaginal tablet daily until symptoms improve (usually 1 week for BV, unpleasant odour and discharge; 1-2 weeks for vaginal dryness, itching, irritation and soreness). ***Prevention:*** ½-1 vaginal tablet 1-7 days per week, as required. Insert one tablet deep into the vagina using a clean finger. Moistening the tablet before insertion may help.

**Contents:** Lactose monohydrate, magnesium stearate and silicon dioxide.

### DO NOT USE if you:

are allergic to milk - contains traces of milk proteins. LadyBalance can be used in cases of dietary lactose intolerance.

### SEE YOUR DOCTOR if you:

have vaginal pain, bleeding or a yellow-green unpleasant smelling

discharge – you could have Trichomoniasis.

### NOT RECOMMENDED to use:

in combination with other vaginal products containing bacteria or yeasts (including milk-based products) - this could reduce the effect of LadyBalance or cause adverse effects.

LadyBalance can be used during pregnancy and breast feeding but see your doctor if you think you have an infection during pregnancy. If symptoms persist, see your doctor.

LadyBalance is for vaginal use only - do not put the tablets in your mouth due to choking hazard.

Keep dry and out of the reach of children.

**Possible side effects:** Clear or gritty discharge (1 in 10 users); light incidents: yeast smell, urge to pass water, headache, itching, spotting (light bleeding) (1 in 100 to 1 in 10 users). These improve after stopping treatment.

### Manufacturer and distributor:

LadyBalance ApS, Karensdalvej 76, DK-7100 Vejle, Denmark.

For more information visit [www.ladybalance.com](http://www.ladybalance.com)

LadyBalance® is a registered trademark.

This product is registered as a Class IIa medical device according to Council Directive 93/42/EEC.

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